

MADE IN
EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

77_2202212_UK

Master Skjold



Article no.	HMI no.	Name - Size	UDI-DI/GTIN
MC 001-5005	30752	Master Skjold - S	5744001971260
MC 001-5010	30753	Master Skjold - M	5744001971277
MC 001-5015	30754	Master Skjold - L	5744001971284

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 24

The Original Nordic
Human Handling Concept



Developed &
Manufactured by:



master-care.dk



Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The product is used to reduce friction from the tires on the wheelchair. This prevents the user from sticking to the rubber and preventing the user's trousers from getting dirty. Must always be removed after transfer.

Correct use of the product:

The product is placed on the wheelchairs tire before transferring to and from the wheelchair. Always remove the product after use.

Warning:

Make sure that the width of the tire is according to the inner width of the product before mounting.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of product is 2-5 years with normal wear and tear. The products life span depends on the use, and load.

Make visual inspections frequently and preferably before every use. Products with broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Master Skjold

Selecting the size:



1. The size of the Master Skjold is selected from the width of the tire.



2. Master Skjold is easily fitted onto the tire. Master Skjold must not slide on the tire or be forced onto the tire. In that case choose a different size - smaller/bigger.

Fitting and use:



3. Place the wheelchair along the edge of the bed.



4. Fit the Master Skjold onto the tire where the buttocks will slide over the tire.



5. The user can now transfer without getting the trousers dirty from the tire's rubber coating.



6. Remove the Master Skjold after use. Otherwise, it can get stuck in the breaks or might spilt.

User Manual - Master Skjold

Technical Specifications

Materials:

ABS

Article no. - Product Name	Size tire width:	Max. User Weight	
MC 001-5005 Master Skjold - S	max. 25 mm	125 kg	
MC 001-5010 Master Skjold - M	max. 31 mm	135 kg	
MC 001-5015 Master Skjold - L	max. 33 mm	135 kg	

WARNINGS



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight:

Recommended max. user weight: 135 kg.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn material. Products with high friction or other deficiencies must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos
Quick Guide etc., visit
www.master-care.dk



Find us on:

